innerspace

THE SIMULATOR COMPANY

Title: **Usage Manual - HTC Vive Pro**

Document type: **Product Documentation**

Document ID: **PD-13**

Author: Martin Geiger

Owner: Martin Geiger

Reviewer(s): Thomas Heeren
reviewed at 2024-08-02 10:48 (UTC +0100)

Approver(s): Marcus Erdmann
approved at 2024-08-02 11:49 (UTC +0100)

Approval date: 2024-08-02

Effective date: 2024-08-02

Next periodic review date: 2025-08-02

Innerspace Simulator

USAGE MANUAL

HTC Vive Pro

LANGUAGES

English	3
German	7
French	11
Spanish	15
Italian	19
Japanese	23
Russian	27

USAGE MANUAL

ENGLISH

1

**Before Simulator
Usage**

2

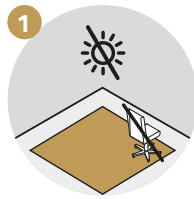
The Simulator Session

3


**After Simulator
Usage**


Before Simulator Usage

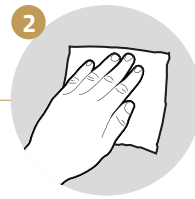
1 Ensure there are no obstacles in the training area, no direct sunlight, and no reflective surfaces.


2 Clean and disinfect the equipment.

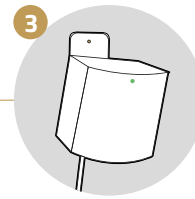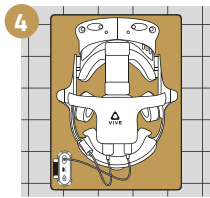
3 Ensure that the base stations are turned on. After a few seconds, the light will turn to green.

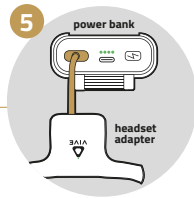
4 Place the headset and power bank on the mat in the middle of the training area.

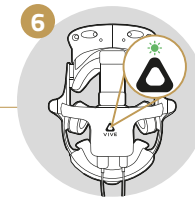
5 Connect the headset adapter to the power bank and switch it on with the button on the power bank.

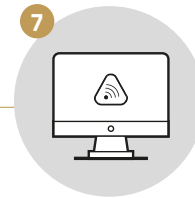
6 A green light appears on the headset adapter.

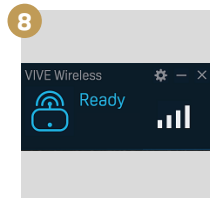
7 Go to the computer and start the VIVE Wireless application.

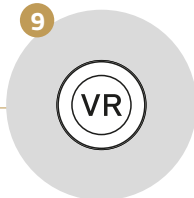
8 Wait until VIVE Wireless is ready. Check if the headset adapter is still active, otherwise re-activate it by clicking the button on top.

9 Start the SteamVR application.

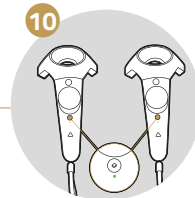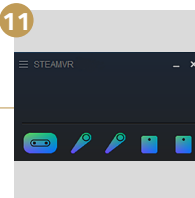10 Switch the controllers on by holding down the button until you hear a beep. A solid blue light will appear on both controllers below this button.

11 Wait until all symbols appear colored. **Note:** The headset and controllers may switch to standby mode. To re-activate them, move them slightly.

12 Start the Simulator Hub application.

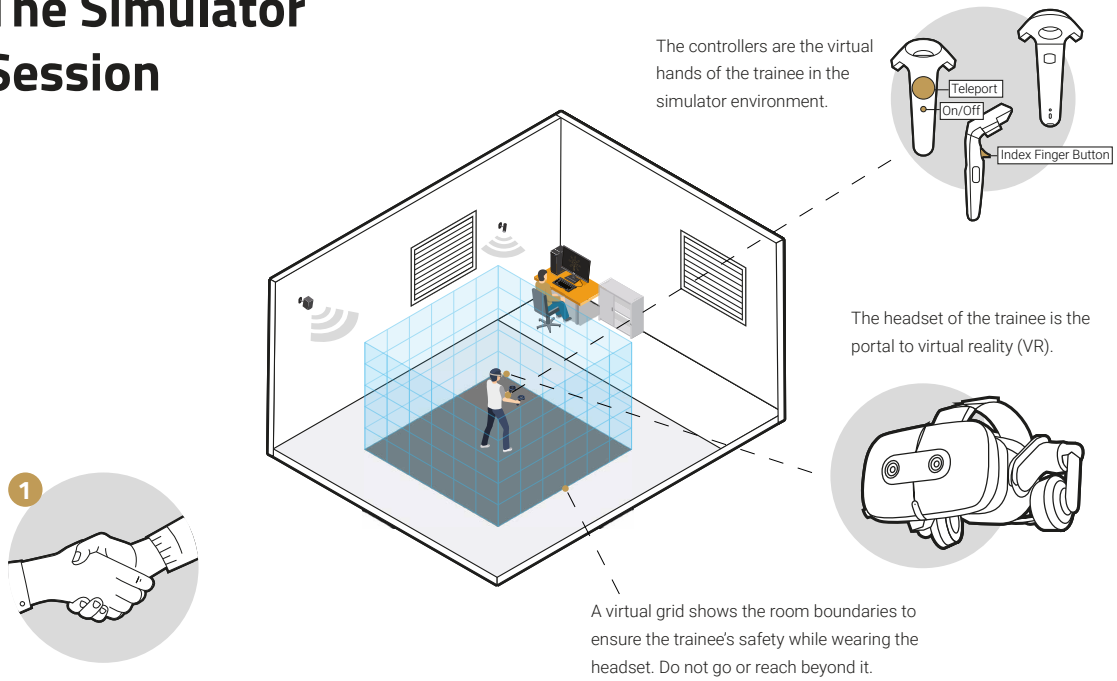
13 Continue with the instructions on the next page.

Questions?

Our troubleshooting guides will help you to solve your issues.

Visit our support portal support.innerspace.eu or contact our helpdesk at support@innerspace.eu


The Simulator Session


Welcome the trainee.

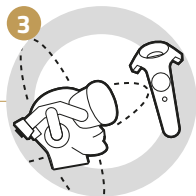
Check if the trainee is aware of the health and safety guidelines.

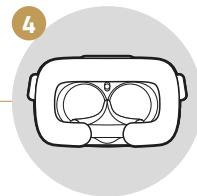
Introduce the trainee to the simulator system and give a brief overview of its content and an introduction to the VR equipment.

Use a new VR hygiene face mask.

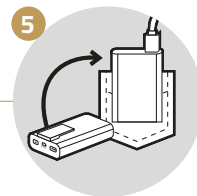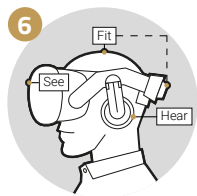
Give the power bank to the trainee and mount it on a belt or pocket.


Help the trainee put on the headset and ensure it is fitted correctly. Let the trainee adjust the headset to achieve a sharper image.

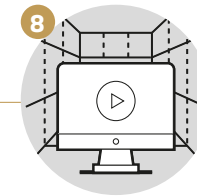Give the controllers to the trainee. The straps must be placed around the wrist and tightened.

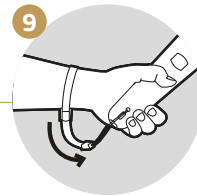
Start the training module on the computer and complete it.


Loosen the straps and take back the controllers from the trainee.

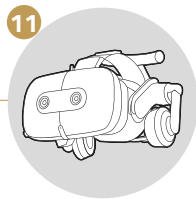
10 Place the controllers on the desk.

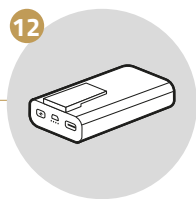
11 Help the trainee take off the headset and welcome them back to the real world.

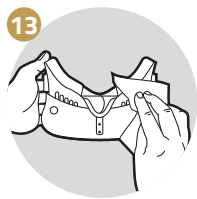12 Take the power bank back from the trainee.

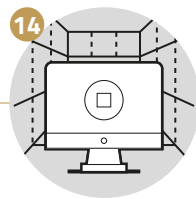
13 Clean and disinfect the VR equipment.


14 Close the training module.

Questions?
Our troubleshooting guides will help you to solve your issues.
Visit our support portal support.innerspace.eu or contact our helpdesk at support@innerspace.eu

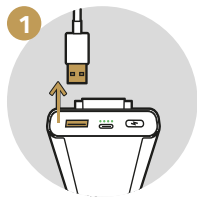
After Simulator Usage

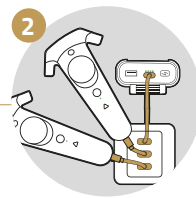1 Disconnect the power bank from the headset adapter.

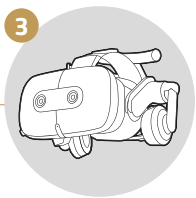2 Connect the power bank and the controllers to the charger.

3 Safely store the equipment in a storage locker. **Note:** Sunlight and moisture may damage the equipment.

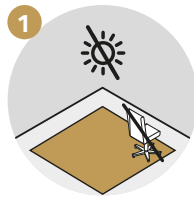
4 Close all applications on the computer, but leave the computer on.

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© Innerspace GmbH
INNERSPACE GmbH • Helga-Krismer-Platz 1 • 6020 Innsbruck • Austria